

HilRom-NaviCare Dome Light Legend



Source	CLINICAL ALERTS	DESCRIPTION	HILL ROM DOME LIGHT	TONE-Dome	TONES-Console	Connect
	Navicare					
Pillow speaker	Normal Call	Normal non-specific patient call	All White/ Slow Flash	NONE	TONE	
	Bathroom request	Patient need to go to BR	Lowest Position (2) yellow/ White/Slow Flash	NONE	TONE	
	Pain Related Request	Patient pressed the pain button	Lowest position (2) Red/White /Slow Flash	NONE	TONE	
Inside Bath	Bathroom Emergency Request	Patient in BR--Pulls Cord for Help	all red Fast Flash	TONE	TONE	
	Shower Emergency Request	Patient in shower-pulls cord-needs help	all red Fast Flash	TONE	TONE	
In room wall panel	Staff Assist	Staff inside the room needs assistance	All Green / Slow Flash	NONE	TONE	
	Staff Emergency	Staff in an emergency situation/ Duress, etc	all red Fast Flash	TONE	TONE	All
	Code Blue	Code Blue situation	All Blue / Fast Flash	TONE	Tone	All
Bed Alarm??	Bed Exit	Patient trying to get out of bed	All Yellow/ Fast Flash	TONE	TONE	
	Patient Equipment (ASBC EQ Jack) Disconnect wall		All Pink/ Fast Flash	TONE	TONE	
	Patient Equipment (ASBC EQ Jack) Bed Disconnect	Bed Disconnected from the wall jack	All white/ Fast Flash	TONE	TONE	Primary RN (vent disconnect)
	Pillow Speaker Disconnect	Pillow speaker disconnected from the wall socket	All Pink/ Fast Flash	TONE	TONE	
	Cord Disconnect	Call cord disconnected	All Pink/ Fast Flash	TONE	TONE	
Doc/EMR	Fall Risk Protocol	Pt. is a fall risk	Middle 4 lights- Light Yellow/ Steady	NONE	NA	
	Pulmonary Protocol	NOT USED	NOT USED	NOT USED		
	Skin Protocol	NOT USED	NOT USED	NOT USED		

LEGEND:

6. PINK LIGHT

- ALL PINK/FAST FLASH=Bed Disconnect from Wall Jack
- ALL PINK/FAST FLASH=Pillow Speaker Disconnected
- ALL PINK/FAST FLASH=Call Cord Disconnected

7. COMBO LIGHT COLORS

- (2) LOWEST LIGHTS YELLOW & WHITE/SLOW FLASH=Patient Bathroom Request
- (2) LOWEST LIGHTS RED & WHITE/SLOW FLASH=Patient Pain Call
- (2) LOWEST LIGHTS YELLOW & GREEN/SLOW FLASH=Patient Falls Risk/Assistance is needed within the room

1. RED LIGHT

- ALL LIGHTS/FAST FLASH=Patient in Bathroom
- ALL LIGHTS/FAST FLASH=Patient in Shower
- ALL LIGHTS/FAST FLASH=Staff Emergency in Patient's room

2. YELLOW LIGHT

- ALL YELLOW/FAST FLASH=Patient Trying to Get Out of Bed
- (4) STEADY MIDDLE LIGHTS=Patient is a Fall Risk

3. BLUE LIGHT

- ALL BLUE/FAST FLASH=Code Blue Situation

4. White Light

- ALL WHITE/SLOW FLASH=Patient in Bathroom

5. STAFF ASSISTANCE

- ALL GREEN/SLOW FLASH=STAFF ASSISTANCE NEEDED IN